

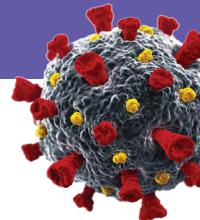
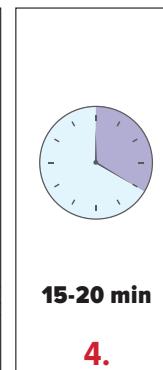
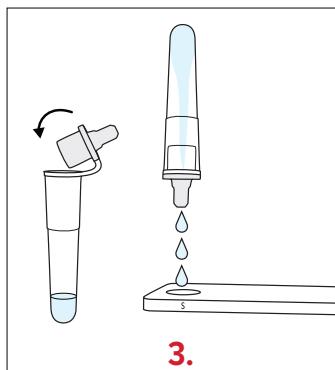


CoviDx™ SARS CoV-2 Rapid Antigen Test

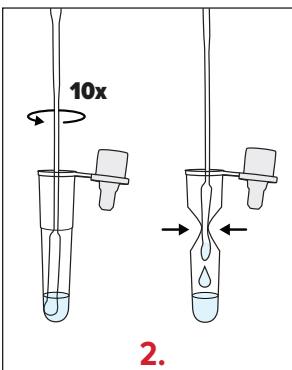

- Qualitative detection of SARS CoV-2 nucleocapsid antigen
- Detects UK, South African and Danish COVID-19 variant strains
- Nasal, oropharyngeal, and nasopharyngeal swabs
- Results in 15 minutes**
- Pre-dispensed extraction reagent
- Room temperature storage
- Provides highly accurate results in a clinically relevant time frame
- Allows optimal patient management
- Enables immediate patient isolation
- Prevents spread of infection
- Improves testing capacity (instrument free)
- Facilitates professionally administered workplace testing and airport screening

SIMPLE PROCEDURE

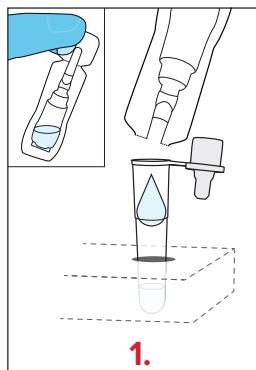

SIMPLE RESULT INTERPRETATION

Positive	Negative
Invalid	

CLINICAL PERFORMANCE

Clinical evaluation of CoviDx™ versus RT-PCR on prospectively collected samples:

	PCR test results		
	Positive (+)	Negative (-)	Total
CoviDx™ results	Positive (+)	45	0
	Negative (-)	3	240
Total number	48	240	288

Sample	PPA	PPA (PCR Ct < 30)	NPA
Oropharyngeal swab	93.8%	100%	100%
Nasopharyngeal swab	93.8%	100%	100%

CoviDx™ – 25 Test Kit: Catalog #CP0016